

FLOWTRON 'HYDROVEN 3' Compression Pump

DVA RAP code AR23

Application:

- The 'Hydroven 3' intermittent pneumatic compression system treats a wide variety of vascular and lymphatic conditions, including venous ulceration, dependent and traumatic oedema and chronic venous insufficiency.
- This device should only be used under the supervision of a qualified medical professional.

Features:

- The 'Hydroven 3' provides clinically effective, non-invasive dynamic compression to the limbs.
- Designed for efficient use in both the clinic or home environment.
- Three chamber leg or arm compression.
- Adjustable pressure control provides tailored therapy to suit individual patients needs.
- The pump has a pressure range of 30-110mmHg, and an inflation/deflation cycle time of 90 seconds each.



- Secure snap-lock connection of all garments to the pump promotes easy set up and prevents accidental disconnection, and single tube attachment to each garment ensures less clutter.
- Automatic detection of single or dual garment attachment makes the system simple to set up. Just 'plug in and switch on' for active treatment.
- Cable retention system ensures compact and easy storage whilst the pump is not in use.

Garments:

- Hydroven's triple-chamber arm and leg garments provide gradient sequential compression, with 10% reduction in pressure in each chamber distally to proximally.
- All garments can be used on either left or right limb, or two units can be used concurrently on both limbs.
- The most often-used leg garment is the half-leg garment, giving full coverage from foot to toe. This comes in one size and 50cm length.
- Full length leg garments are all one size, but are available in 5 lengths, ranging from 66 to 92 cm, according to the patient's leg length.
- Arm sleeves are available in 2 lengths, which comfortably accommodate most users.
- For those with larger limb circumference, there are insert pieces available for each garment, which are easily applied by zip closure.
- Due to the generous sizing range of the garments, insert pieces are not usually required.

Product	Code	
'Hydroven 3' Pump	510009AU	
Garments	Limb Garment	Inserts
Half Leg 50cm	5103L50	5103LI50
Full Leg 66cm	5103L66	5103LI66
Full Leg 71cm	5103L71	5103LI71
Full Leg 76cm	5103L76	5103LI76
Full Leg 84cm	5103L84	5103LI84
Full Leg 92cm	5103L92	5103LI92
Full Arm 68cm	5103A68	5103AI68
Full Arm 78cm	5103A78	5103AI78

'COMPRESS' Sequential Compression Pump Kit – Complete


DVA RAP code AR 23

Application:

- This device may be prescribed for post phlebotic syndrome, varicose veins, varicosities of pregnancy, leg ulcers, venous ligation, knee/hand surgery, prevention of DVT, Raynauds disease, arteriosclerosis, gravitational oedema, lymphatic oedema, post-mastectomy oedema, after cast removal, flexion contractions.
- This device must only be used under the supervision of a qualified medical professional.

Features:

- Ideal for home use, the Compress DL2002D Sequential Compression Kit includes a pump, all required sleeves plus tubes and connectors.
- The Clinic model includes two full-length leg sleeves, an arm sleeve, plus all tubes and connectors.

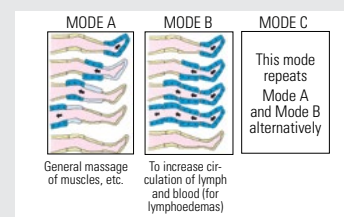

- The Compress DL2002D Sequential Compression Pump provides highly effective, non-invasive treatment using air compressed on to the limb to move fluids.
- 'Compress' works with a gentle peristaltic action, massaging the limb and effectively aiding the management of oedema due to venous insufficiencies and lymphoedema.
- This pump is easy to operate, with 2 controls – one for the required pressure level, and the other for the timing of each treatment.
- Features easy-to-read digital pressure readout gauge.
- There are 3 modes of compression at the touch of a button. The first mode ('A') inflates and then deflates each of the 4 chambers, one at a time from foot to thigh. This mode is suitable for general muscle massage. The second mode ('B') inflates each chamber sequentially, foot to thigh, until all chambers are inflated, then deflates all 4 chambers. This mode increases circulation of lymph and blood. The third mode ('C') alternates between 'A' and 'B'.

Product	Code
Compress Compression Pump Kit - Complete	DL2002D
Leg replacement sleeve	DL - LEG
Arm replacement sleeve	DL - ARM