

DALE Dale Abdominal Binder



Features:

- single ply stretch loop elastic (latex-free)
- velcro closure anywhere on belt for perfect fit, and can be angled to conform to abdominal contour
- provides full, even pressure and aids abdominal muscle
- colour white
- sizing: available in either 23cm (pictured) or 30cm widths.

Application:

- suitable for post-surgical / incisional hernia
- maternity and general abdominal support


Product care:

- join the hook and loop fasteners before washing
- hand wash in warm water using mild detergent
- dry out of direct sunlight

23cm width	30cm width	Size	Circum
DO 410	DO 810	Regular	to 114
DO 411	DO 811	Large	114-157
DO 418	DO 818	X Large	157-191

Sizing: measure circumference of hips.

DALE Dale Abdominal Lumbar Support

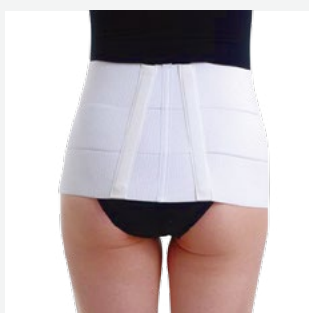

Features:

- single ply stretch loop elastic (latex-free)
- the same product as the Dale Abdominal Binder, but with two removable aluminium supports which can be hand-shaped to hip curvature
- velcro closure anywhere on belt for perfect fit, and can be angled to conform to abdominal contour.
- colour white
- sizing: available in either 23cm (pictured) or 30cm widths.

Application:

- aids abdominal muscle while providing excellent general lumbar-sacral support

Product care:

- remove aluminium stays, and join the hook and loop fasteners before washing
- hand wash in warm water using mild detergent
- dry out of direct sunlight

23cm width	30cm width	Size	Circum
DO 442	DO 842	Regular	to 114
DO 443	DO 843	Large	114-157
DO 448	DO 848	X Large	157-191

Sizing: measure circumference of hips.



Oppo 2062 maternity back support



Features:

- helps reduce lower back pain and fatigue during pregnancy
- helps relieve pain by gently lifting the abdomen
- provides compression and support to the hips and lower back
- breathable elastic and soft pile bonded to foam
- 50% cotton

Application:

- for lower back pain and fatigue during pregnancy

To apply:

- position mid section of support on lower back
- fasten front panels just below belly for support
- pull side straps and secure for extra compression

Product Code	Size	Circ cm
OPP2062S/M	small/medium	To 106
OPP2062L/XL	large/x large	106-110
OPP2062XXL	xx large	111 +

Sizing: measure circumference of hips.



Oppo 2262 stability belt



Features:

- may be used post-partum and following traumatic injury
- relieves instability and stress through compression of the sacroiliac joints
- reinforcement strap provides additional compression
- hook and loop closure for easy application

Application:

- relieves instability and stress through compression of the sacroiliac joints.

To apply:

- position belt just below belly for support
- bring straps to back and fasten securely
- pull side straps to adjust tightness then fasten

Product Code	Size	Circ cm
OPP2262XS	x small	71.1-86.7
OPP2262S	small	86.7-91.8
OPP2262M	medium	91.8-96.8
OPP2262L	large	96.8-101.9
OPP2262XL	x large	101.9-109.5

Sizing: measure circumference around hips.



Oppo 3462 SIJ support


Features:

- breathable material to reduce heat
- may be worn during any daily activity
- extra elastic strap helps enhance core stabilization
- MAXWRAP® inner lining with silicone prevents slipping

Application:

- Hypomobility of SI joint
- Reposition pelvic alignment after birth
- Sacroiliac joint pain
- Sacroiliac joint instability
- Coccydynia

To apply:

- position belt just below belly for support and bring straps to back and fasten securely
- use the additional elastic strap to apply compression
- secure the fixed side first, pull over the strap
- position the outer strap over the inner strap. For more compression, pull the outer strap across the inner strap to tighten

Product Code	Size	Circ cm
OPP3462XS	x small	66.0 - 79.4
OPP3462S	small	79.4 - 94.0
OPP3462M	medium	94.0 - 107.3
OPP3462L	large	107.3 - 120.0

Sizing: measure circumference of hips.



Oppo 4062 maternity belt


Features:

- helps with proper posture and relieves stress to the body by encouraging proper body position and movements
- soft pile bonded to foam

Application:

- helps relieve pain during pregnancy by gently lifting the abdomen


To apply:

- position front panel just below belly for support
- bring straps to back and fasten

Product Code	Size
OPP4062	One size fits most



Rehband 1041 dosi symphysiolysis belt

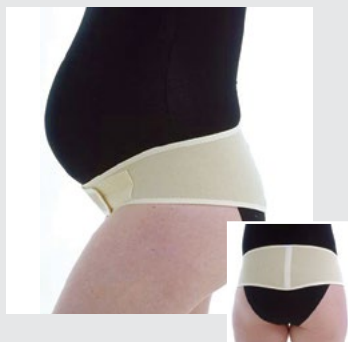
Features:

- elastic symphysiolysis support of soft material with a pre-shaped fit
- provides pain relief and increased proprioception

Application:


- used for sacroiliac joint dysfunction and symphysiolysis

Product Code	Size	cm
REH1041XS	x small	79 - 86
REH1041S	small	86 - 93
REH1041M	medium	93 - 100
REH1041L	large	100 - 107
REH1041XL	x large	107 - 112
REH1041XXL	xx large	112 - 119

Sizing: measure circumference around hips



Rehband 1045 dosi sacro belt

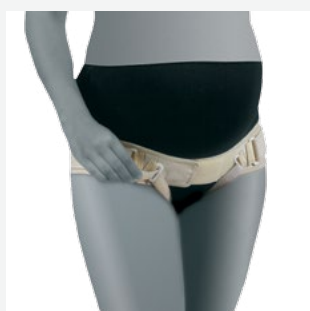
Features:

- offers pain relief and increased proprioception
- elastic symphysiolysis support in a soft material with a pre-shaped fit
- back plate of a non-elastic material offering stabilization
- stabilizing strap at each side for individual adjustment and support
- removable crotch straps are included
- the combination of elastic and non-elastic materials provides stability and fit

Application:

- used for sacroiliac joint dysfunction and symphysiolysis

Product Code	Size	cm
REH1045XS	x small	79 - 86
REH1045S	small	86 - 93
REH1045M	medium	93 - 100
REH1045L	large	100 - 107
REH1045XL	x large	107 - 112
REH1045XXL	xx large	112 - 119

Sizing: measure circumference around hips

Pre-birth cradle



Features:


- significantly reduces lower back ache associated with pregnancy
- completely adjustable, and suitable throughout entire pregnancy
- may be worn against skin, or over singlet
- Colour: white

Application:

- support for abdomen, lower back and pelvis during pregnancy

Product Code	Size
B530	One size fits most

Barrere 384 pelvic / sacroiliac strap

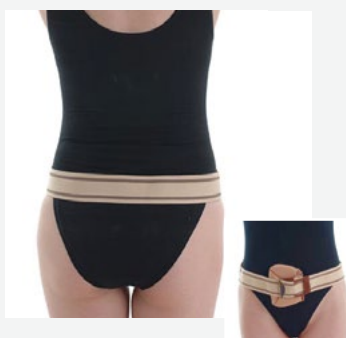

Features:

- 5cm firm elastic strap all around
- gives firm compression to the sacral joint and relieves pressure on the pubic bone
- strong, surgical elastic webbing
- slide buckle and pad at front
- also available in 2 other models:
 - 385 – 5cm strap at front, with 10cm back pad
 - 386 – 5cm strap at front, with 15cm back pad
- the wider padded back plates in the 385 and 386 models provide additional lower back support
- larger sizes can be manufactured on request


Application:

- commonly used to relieve sacral pain during pregnancy

Product Code	Size	Circ cm
B384 XS	x small	81 - 86
B384 S	small	86 - 91
B384 M	medium	91 - 96
B384 L	large	96 - 101
B384 XL	x large	101 - 106
B384 XXL	xx large	106 - 111

Sizing: measure circumference around hips

Barrere 385 sacroiliac/10cm lumbar support

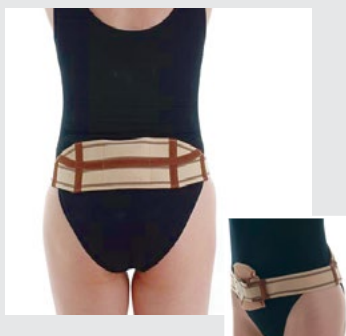

Features:

- 5cm strap at front, with 10cm back pad
- gives firm compression to the sacral joint and relieves pressure on the pubic bone
- back pad provides lower back support
- strong, surgical elastic webbing
- slide buckle and pad at front
- also available in 2 other models:
 - 384 – 5cm elastic pelvic strap all around (no back pad)
 - 386 – 5cm front, with wider 15cm back pad for additional lower back support
- larger sizes can be manufactured on request


Application:

- commonly used to relieve sacral & back pain during pregnancy

Product Code	Size	Circ cm
B385 XS	x small	81 - 86
B385 S	small	86 - 91
B385 M	medium	91 - 96
B385 L	large	96 - 101
B385 XL	x large	101 - 106
B385 XXL	xx large	106 - 111

Sizing: measure circumference around hips

Barrere 386 sacroiliac /15 cm lumbar support

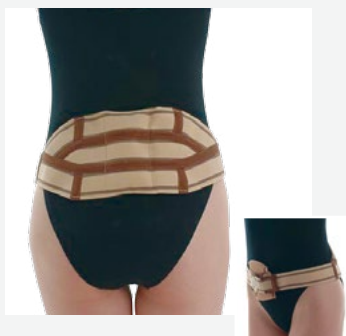

Features:

- 5cm strap at front, with 15cm back pad
- gives firm compression to the sacral joint and relieves pressure on the pubic bone
- back pad provides lower back support
- strong, surgical elastic webbing
- slide buckle and pad at front
- also available in 2 other models:
 - 384 – 5cm elastic pelvic strap all around (no back pad)
 - 385 – 5cm front, with narrower 10cm back pad
- larger sizes can be manufactured on request

Application:

- commonly used to relieve sacral & back pain during pregnancy.

Product Code	Size	Circ cm
B386 XS	x small	81 - 86
B386 S	small	86 - 91
B386 M	medium	91 - 96
B386 L	large	96 - 101
B386 XL	x large	101 - 106
B386 XXL	xx large	106 - 111

Sizing: measure circumference around hips