

Drop foot ankle foot orthosis open - Open design

DVA RAP code

DVA RAP code

DVA RAP code

AR19

AR19

AR19



Features:

- limits plantar flexion, common peroneal nerve injuries.
- · can be adjusted for a custom fit by heating, cutting and
- · available in left or right fitting

Application:

indicated for foot drop

Left	Right	Size	Female	Male
AUPHYAFOXS-L	AUPHYAFOXS-R	XS	4-6	4-5
AUPHYAFOS-L	AUPHYAFOS-R	S	6-8	5-7
AUPHYAFOM-L	AUPHYAFOM-R	М	8-10	7-9
AUPHYAFOL-L	AUPHYAFOL-R	L	10-13	9-12

Sizing by US shoe size.

Drop foot ankle foot orthosis



- · designed to hold the foot correctly
- · supports the foot to stop dragging
- fits easily into shoe, heat mouldable, easily trimmed
- shoe size: metric

Application:

indicated for foot drop

Left	Right	Size	Shoe size (metric)
B770L S	B770R S	S	To 38
B770L M	B770R M	M	39 - 43
B770L L	B770R L	L	44 - 46
B770L XL	B770R XL	XL	47+

Sizing according to European (metric) shoe size.



Foot-up drop foot orthosis



Shoe Model



Shoeless Wrap Model

Features:

- Foot-up is a lightweight ankle-foot orthosis offering dynamic support for drop foot or similar complaints
- · provides visible improvement in gait by supporting the foot the moment it is raised
- cushioned ankle wrap is made from a breathable, 3-layer material that can be worn comfortably for long periods
- plastic inlay fits discreetly between the tongue and laces of the shoe and is practically invisible when worn
- · a foot accessory (circled) is also available which attaches to the shoe model to allow wear without footwear.

Application:

dynamic support for drop foot or similar complaints for which support of dorsiflexion is desirable

Product Code	Size	(cm)			
Shoe Model	Shoe Model				
Measure circumferer	Measure circumference just above ankle bone				
OSS07810-1	M	18-21			
OSS07810-2	L	22-26			
OSS07810-3	XL	27-33			
Shoeless-Wrap Accessory					
Measure circumferer	nce around foot at ar	ch/instep			
OSS27810-1	S	Less than 20			
OSS27810-2	M	20-24			
OSS27810-3	L	24-28			
OSS27810-4	XL	28 +			

Valgus 4050 arch/metatarsal support

Leather



Features:

- anatomically designed moulded rubber with leather cover
- metatarsal dome
- available in narrow and wide fittings
- order by shoe size (metric) and width
- supplied in pairs

Application:

· arch and metatarsal support

Narrow	Wide	Shoe Size (European)
B1801/100/33		33
B1801/100/34		34
B1801/100/35		35
B1801/100/36		36
B1801/100/37		37
B1801/100/38	B1801/200/38	38
B1801/100/39	B1801/200/39	39
B1801/100/40	B1801/200/40	40
B1801/100/41	B1801/200/41	41
B1801/100/42	B1801/200/42	42
	B1801/200/43	43
	B1801/200/44	44
	B1801/200/45	45

B1801/200/46 46

DVA RAP code AR04

Sizing according to European (metric) shoe size.



AR04

DVA RAP code

DVA RAP code AR04

DVA RAP code AR04

Metatarsal 1920 support



Features:

- thin, flat leather support with wide metatarsal support
- · order by shoe size (metric)
- · supplied in pairs

Application:

· metatarsal support

Product Code	Shoe Size (European)
B1920/35	35
B1920/36	36
B1920/37	37
B1920/38	38
B1920/39	39
B1920/40	40
B1920/41	41
B1920/42	42
B1920/43	43
B1920/44	44
B1920/45	45
B1920/46	46

Sizing according to European (metric) shoe size



Tulis 740 standard heel cups

Silicone gel



Features:

- · double ribbed classic heel cups provide relief from heel strike
- · worn in shoes
- available as pairs
- size by weight (kg)

Application:

• helps prevent sore heels and knees, shin splints and heel spurs

Product Code	Size	Sizing: weight (kg)
B740 R	regular	under 80
B740 L	large	over 80

Sizing according to weight (kg)



Tulis 741 heavy duty heel cups

Silicone gel



Features:

- · double ribbed heavy duty heel cups
- two level rib pattern for higher level shock absorption
- per pair
- size by weight (kg)
- small size for young athletes

Application:

excellent for active people or those with chronic heel pain

Product Code	Size	Sizing: weight (kg)
B741 S	S	up to 40
B741 R	M	40 - 80
B741 L	L	over 80

Sizing according to weight (kg)



Tulis 743 fatigue mats

Silicone gel



- TuliGEL Heel Fatigue Mats are the ideal protection when a lowprofile shock absorber is needed
- slipping smoothly into the heel of the shoe TuliGEL Heel Fatigue Mats have the same protective design as TULI'S
- · universal (one size fits all)

B743 Universal size

AR04

DVA RAP code



TIII 1'5

Tulis 744 metatarsal cushions

DVA RAP code

AR04

Silicone gel



Features:

- for protection during normal daily routines
- · waffle design assists in pain relief when wearing fashion or dress shoes
- · constructed of high impact TuliGEL
- per pair
- one size fits all

ι	Jniversal	size
	ι	Universal



Tulis 745 heel fatigue mats

Silicone gel



Features:

- · tuliGEL Personal Fatigue Mats have been scientifically and medically designed to protect feet from hard, unyielding surfaces
- the unique tic-tac-toe soft waffle pattern interrupts the shock waves created while walking or running, helping to prevent fatigue, tired, swollen, aching feet and backache
- if you work on hard concrete surfaces, TuliGEL Personal Fatigue Mats can make a real difference in how you feel
- universal (one size fits all)

DVA RAP code AR04

DVA RAP code AR04

B745 Universal size



Tulis 107 diamondbacks heel/arch supports

Silicone gel



Features:

- a very comfortable heel and arch support
- combination of soft gel with a supportive foam arch
- thin, easy-fitting design
- will fit most shoe styles, including dress shoes

Product Code	Size	Female	Male
B107 S	Small	5-7	3-5
B107 M	Medium	8-11	6-9
B107 L	Large	12-16	10-14

Sizing by US shoe size.

Body Assist

Heel spur heel cups

Silicone ael



Features:

- Silicone Medical Gel provide exceptional heel support, protection & comfort.
- ideal for sports use, gardening, factory staff, construction workers, office staff & for people who stand on hard surfaces
- G.C. Heel Spur Pads are made from a unique Medical Silicone Gel compound that assists to dissipate shock & shear, caused by impact to the heel & foot
- they emulate the fatty tissue, usually found on the underfoot.
- · damaging stress forces normally absorbed by the body's musculo-skeletal system, are now absorbed by the pad & away from the body
- · fits easily into existing footwear
- suitable for Men or Women
- · washable & hygienic

Application:

 designed to support & comfort the heel &/or associated heel spurs conditions

DVA RAP code

Product Code	Size	Shoe size
NHGC66HC1	S	37- 40
NHGC66HC2	M	40.5-42
NHGC66HC2	L	42.5 +

Sizing by European shoe size.

CALL: 1300 723 092



DVA RAP code

DVA RAP code

AR04

AR04

BodyAssist

Heel spur pads

Silicone gel



Features:

- these Heel Spur Pads gently cushion the heel, while dissipating shock and shear generated by heel strike.
- a removable 'tear drop' central plug allows the heel centre to be suspended around a supportive ring of silicone gel.
- the pads fit snugly into the heel of a shoe.
- shock attenuation properties of this material have been proven effective in modifying heel strike trauma

Application:

 clinically designed to assist in reducing pain associated with Heel Spurs

Product Code	Size	Shoe size
NHGC66SP1	S	To 38
NHGC66SP2	M	38.5 - 42
NHGC66SP2	L	42.5 +

Sizing by European shoe size

Body Assist

Gel shoe insole

Silicone gel



Features:

- · provides overall support , protection, comfort & help reduce painful shock waves to the body & lighten the load on joints.
- · ideal for sports use, gardening, factory staff, construction workers, office staff & for people who stand on hard surfaces.
- made from a unique Medical Silicone Gel compound that assists to dissipate shock & shear, caused by impact to the heel & foot.
- they emulate the fatty tissue, usually found on the underfoot.
- damaging stress forces normally absorbed by the body's musculo-skeletal system, are now absorbed by the footbeds & away from the body.

Application:

- · suitable for managing minor cases of splayfoot.
- · fits into existing footwear.
- suitable for Men or Women
- · washable & hygienic

Product Code	Size	Shoe size
NHGC66FB0	XS	37- 38
NHGC66FB1	S	39- 41
NHGC66FB2	M	41- 43
NHGC66FB3	L	44- 46
NHGC66FB4	XL	47 +

Sizing by European shoe size

Trio Heel elevators



Features:

- trio elevators come in different heights and should be placed under the heel area of the orthotic.
- · choose between sizes small, medium, or large.
- they are manufactured from extra firm density to prevent wearing out or crushing.
- available in 4, 6 and 8mm rises


Application: adding the Trio Elevator (heel lift) can help reduce the stress on the achillies tendon and relieve pain associated with Achilles tendonitis and Sever's disease.

DVA RAP code AR04

· when a structural short leg is present an Elevator can

4mm	6mm	8mm	Size	Shoe size
TALHE4S	TALHE6S	TALHE8S	S	To 37
TALHE4M	TALHE6M	TALHE8M	М	38- 43
TALHE4L	TALHE6L	TALHE8L	L	44 +

Sizing by European shoe size

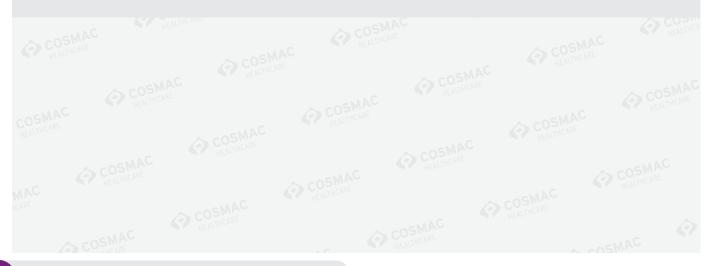



DARCO Darco peg-assist

DVA RAP code

AR04



Features:

- · designed for use in most DARCO square-toe shoes, other than the DARCO Med-Surg Shoe (for which DA-PTQ Insoles should be used)
- removable pegs allow for localized off-loading of wounds and ulcerations of the foot
- pressure reduced by as much as 60% while allowing patient to remain ambulatory.
- poron® cover eliminates the incidence of ring edema and edge abrasion.
- included stabilizer board prevents adjacent pegs from collapsing.
- available in a wide range of sizes and configurations

Application:

 The DARCO PegAssist™ Insole System features a multi-indication removable-peg chassis that effectively off-loads the plantar aspect of the foot after surgery or when wounds are present

Product Code	Size	Female Shoe Size	Male Shoe Size
DA-PQ0	XS	< 5	
DA-PQ1	S	5.5 - 8	
DA-PQ2	М	8.5 - 10	6 - 8
DA-PQ3	L	10.5 - 13	8.5 - 11
DA-PQ4	XL	13.5+	11.5 - 14

Sizing by US shoe size

Darco peg-assist insole



Features:

- designed for use with the DARCO Med-Surg Shoe.
- · removable pegs allow for localized off-loading of wounds and ulcerations of the foot
- · pressure reduced by as much as 60% while allowing patient to remain ambulatory.
- poron® cover eliminates the incidence of ring edema and edge abrasion.
- · included stabilizer board prevents adjacent pegs from collapsing.
- · available in a wide range of sizes and configurations

Application:

 The DARCO PegAssist™ Insole System features a multi-indication removable-peg chassis that effectively off-loads the plantar aspect of the foot after surgery or when wounds are present.

DVA RAP code


Male	Size	Shoe Size
DA-PTQM1	S	6 - 8
DA-PTQM2	M	8.5 - 10
DA-PTQM3	L	10.5 - 12
DA-PTQM4	XL	12.5 - 14
Female	Size	Shoe Size
DA-PTQW1	S	4 - 6
DA-PTQW2	M	6.5 - 8
DA-PTQW3	L	8.5 - 10

Sizing by US shoe size



Oppo 1408 achilles brace w/silicone pad



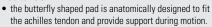

- the concentric ripples on the pad provide intermittent massage-like compression, to effectively reduce edema, relieve spasm and speed up the recovery process.
- · adjustable crisscross straps control the range of foot plantar-flexion to shorten the achilles tendon to relieve pain and inflammation.
- CoolPrene® material provides shock absorption, dynamic compression and better breathability on the injured ankle over prolonged use.

Application:

- achilles bursitis
- · achilles tendonitis
- · post-operative use

To Apply:

• open the brace and place the pad under the sole to insert your foot through the brace.

DVA RAP code

AR02

- · pull the lower straps. Make sure the pad is correctly positioned below the anklebone.
- plantar flex your foot and secure the upper closure.
- · keep your foot in this position and pull the lower straps across the back of the brace. Fasten the straps on both sides of the lower leg.
- make sure the brace is positioned correctly, below the anklebone

Left	Right	Size	cm
OPP1408S-L	OPP1408S-R	S	15.6 - 20.6
OPP1408M-L	OPP1408M-R	М	20.6 - 25.7
OPP1408L-L	OPP1408L-R	L	25.7 - 30.8
OPP1408XL-L	OPP1408XL-R	XL	30.8 - 35.9

Measure circumference just above ankle bone.

DVA ENQUIRIES:



& McDavid McDavid A436 achilles tendon strap

AR02 **DVA RAP code**



Features:

- applies constant mild compression at the achilles tendon through the use of a unique tubular buttress
- · helps relieve some dynamic forces caused by constant running & jumping
- · adjustable 2 way stretch neoprene strap with hook and loop closure

Product Code	Size	Shoe Size
NHA4361	S - M	To 10
NHA4363	L - XL	10 +

Sized on mens US shoe size



Achilles 1400 heel protection sleeve

DVA RAP code AR02



Features:

- the soft, comfortable gel pad protects the posterior of the heel from friction while gently compressing and reducina pressure.
- washable and reusable.
- · slim profile design fits most footwear.

Application:

 ideal for relief from pump bumps (Haglund deformity), or sore, fatigued or injured achilles tendon

AUGEL1400M One size Small/Medium



Adjustable night splint

DVA RAP code

AR02



Features:

- · patented Formfit 3D molded technology allows for variable padding thickness in strategic locations for superior comfort and fit
- · soft lycra lined foam interior is plush and breathable, eliminating heat build-up
- · three lycra lined foam padded straps with user friendly centre release buckles ensure immobilisation while maintaining comfort across the flexor tendons
- · adjustable dorsi-flexion straps provide a gentle stretching of the plantar fascia and achilles tendon (adjustable from 90 to 10 degrees dorsi-flexion)
- · lightweight, low profile shell is sturdy and breathable
- · available in Men's and women's, three sizes
- measure as per shoe size

Application:

ideal for plantar fascitis, achilles tendonitis and other lower extremity overuse injuries


Product Code	Size	Female Shoe Size	Male Shoe Size
AUPHYNSPS	S	To 7.5	To 7
AUPHYNSPM	М	8-10.5	7.5-10
AUPHYNSPL	L	11 +	10.5 +

Sized on mens US shoe size



Exoform dorsal night splint



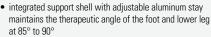

- thermoformed Form Fit® padding on the inside of the soft good improves patient comfort
- full calf and heel strapping holds the foot in the desired angle and eliminates rotation and migration of the product
- flex Edge™ overmolding eliminates pressure points and discomfort while protecting sleeping partners
- · semi-rigid plantar pad reduces discomfort along the sides of the foot while providing an improved angle of dorsiflexion

• non-slip surface on plantar pad to help prevent slipping

DVA RAP code

AR02

· aesthetically superior, low-profile design

Application:

- plantar Fasciitis
- · achilles Tendonitis
- · drop foot
- post-static pain

Product Code	Size	Shoe Size
OSS50085	М	To 42
OSS50087	L	43 +

Sizing by European shoe size





Hallux valgus 6036 night splint

DVA RAP code AR02



Features:

- the cupped end of the plastic splint cradles the big toe, separating it from the second toe, forcing a more straightened position.
- also providing optimal post-operative capsular relief following corrective surgery
- the anatomically adapted shape of the splint aids the physiological correction of the basic joint of the big toe.
- two adjustable Velcro fasteners allow precise adjustment the solint
- · specify left or right foot

Application:

 designed to correct, over time, inward deformity of the big toe (hallux valgus)

Left	Right	Size	Shoe Size
6036SL	6036SR	S	34-37
6036ML	6036MR	М	37-40
6036LL	6036LR	L	41-46

Sizing by European shoe size.

⊗ апевнэ

Hallux valgus 510 night strap



AR02



Features:

- a light and flexible strap to unload the joints of the big toe
- especially designed for nightly or post-operative usage
- padded with skin-caring and breathable material
- · provides pain relief and guides the big toe
- universal size specify left or right foot

Application:

· designed to help correct, over time, inward deformity of the big toe (hallux valgus)

DVA RAP code

Left	Right	Size
REH510L	REH510R	universal

О апевнээ

Hallux valgus 509 day strap

DVA RAP code AR02



Features:

- a light and flexible strap to unload the joints of the big toe
- · padded for optimal comfort
- · can be used with or without shoes
- gives pain relief and may be a condition for a normalised function of the muscles, guides the big toe
- universal size specify left or right foot

Application:

· designed to help correct, over time, inward deformity of the big toe (hallux valgus)

Left	Right	Size	
REH509L	REH509R	universal	

DARCO Darco toe aligment splint

DVA RAP code AR02



Features:

- low profile metatarsal band fits easily into most footwear and reduces slippage
- · soft toe straps will not irritate skin
- special T-Strap gently guides toe(s) into proper alignment for post-procedural healing
- one sizes fits all
- latex Free
- · colour- White

Application:

• a cost-effective solution providing superior post operative toe alignment after Hallux Valgus, Hammer Toe and Tailor's Bunion procedures

Product Code	Size
DA-TAS	universal

DVA ENQUIRIES: