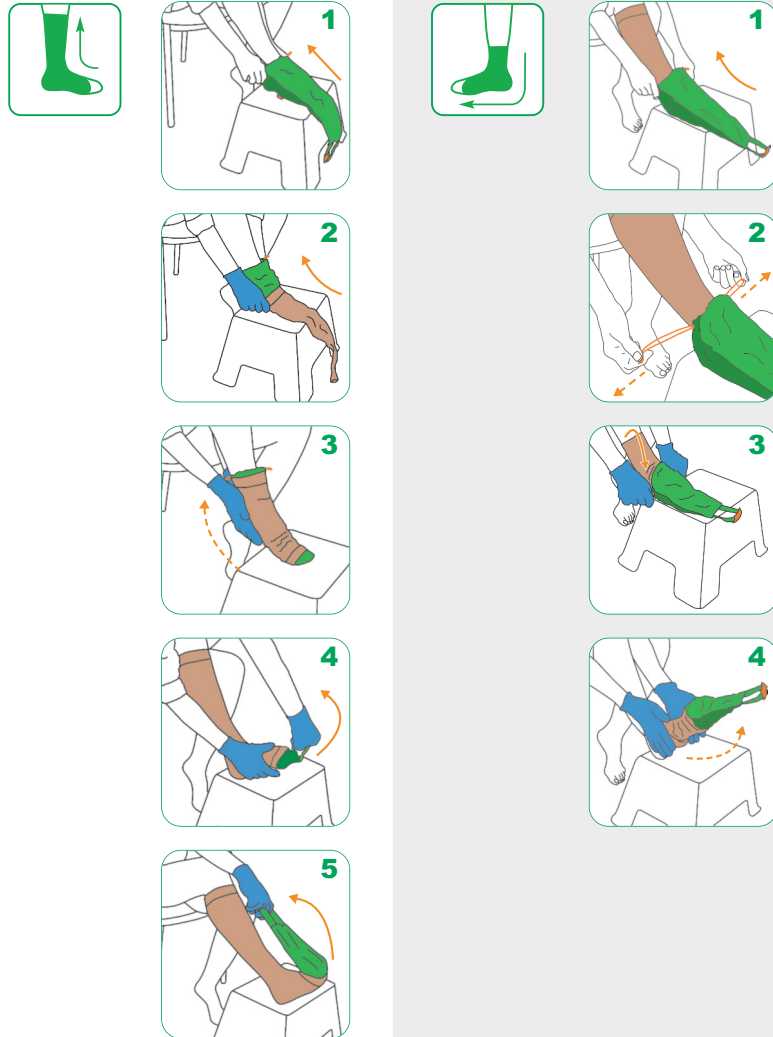
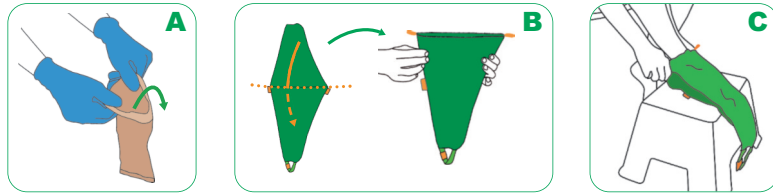
ARION Sim-Slide®

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Arion Sim-Slide®

Donning and doffing aid for medical compression stockings with an open toe

Important information before you start using the product:

- A.** Useful tips:
 - Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.
 - In case the compression garment has a silicon border, make sure to turn it outwards.
- B.** Make sure the product is ready for use:
 - Fold back the extra fabric into the „green bag“
 - Align the fold line with the top of the orange labels
- C.** If you have difficulty to reach your toes, please place your foot on a low chair to avoid bending too much.

On

Donning instructions:

1. Place the Sim-Slide® across the foot. To prevent the Sim-Slide® from gliding off the foot, gently pull on the orange tabs.
2. Pull the stocking over the Sim-Slide®
3. Slide the stocking over the foot, so that the heel of the stocking fits into the correct position
4. Place one hand firmly on the heel and with the other hand pull the loop of the Sim-Slide®
5. Gradually pull the Sim-Slide® out of the stocking

Off

Doffing instructions:

1. Slide the Sim-Slide® over the stocking
2. Firmly pull on the orange tabs so that the Sim-Slide® sits just above the ankle
3. Fold the stocking over the Sim-Slide® as far as possible
4. Starting at the heel, gradually slide the stocking and the Sim-Slide® from the foot

Watch our **video instructions** on uqr.to/SimEN

SIZING

REF	Size	Shoe size EU	Shoe size UK		Shoe size US	
			women	men	women	men
E0SL01	Small	≤ 34 - 36	≤ 1 ¹ / ₂ - 3 ¹ / ₂	≤ 2 ¹ / ₂ - 4	≤ 4 - 6	3 - 4 ¹ / ₂
E0SL02	Medium	37 - 38,5	4 - 5 ¹ / ₂	4 ¹ / ₂ - 6	6 ¹ / ₂ - 8	5 - 6 ¹ / ₂
E0SL03	Large	39 - 41	6 - 7	6 ¹ / ₂ - 7 ¹ / ₂	8 ¹ / ₂ - 9 ¹ / ₂	7 - 8
E0SL04	Extra large	≥ 42	≥ 7 ¹ / ₂	≥ 8	≥ 10	≥ 8 ¹ / ₂

Watch our **instruction video** here: uqr.to/SimEN or scan this QR code:

More information on www.slide-solutions.com

Sim-Slide® from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. The friction reduction is particularly helpful to slide the garment over the heel area, which is the broadest part of the foot.

EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

COMPOSITION

- Product: Polyamide 92%, Polyurethane 3%, Silicone 2%, Polyester 2%, Rubber 1%
- Package: Polypropylene (PP)

CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

⚠ WARNINGS

- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

IMPORTANT

- Please **read the instructions for use carefully**. If you have any questions, please contact your reseller.
- **Please keep the package and the instructions for use during the time of use of the product.**
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- **Use the product on a single person.** If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

STORAGE

Keep away from direct sunlight or other heat sources.

CARE INSTRUCTIONS

- Wash by hand, at max. 30°C/86°F.
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

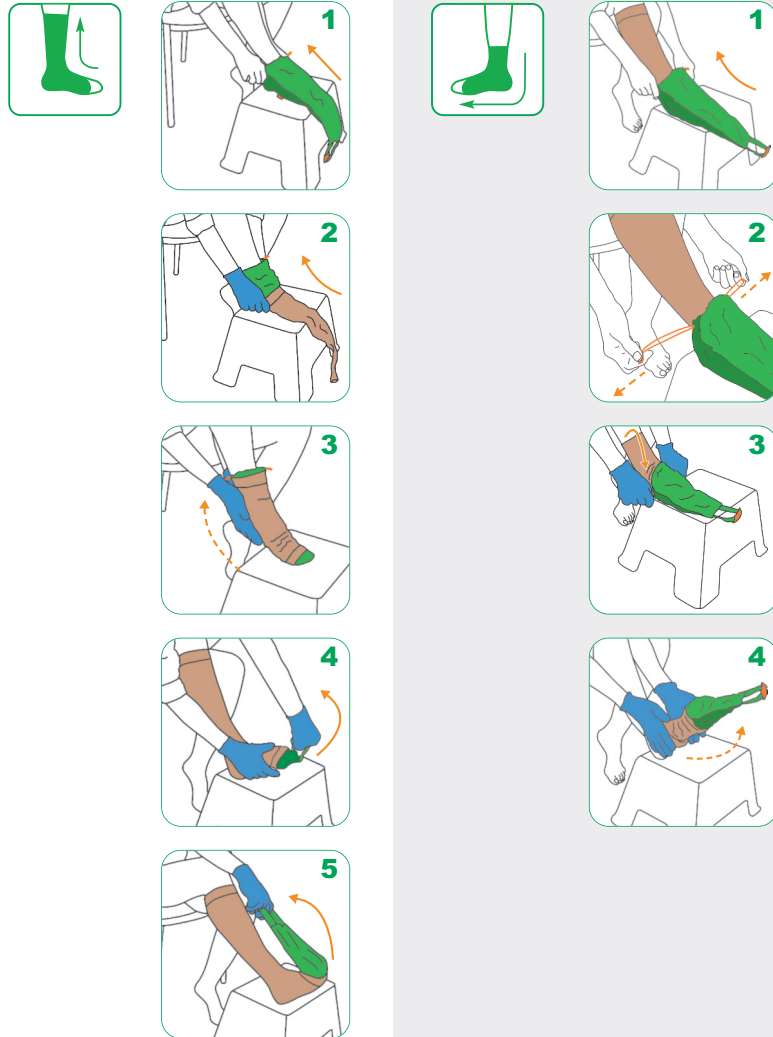
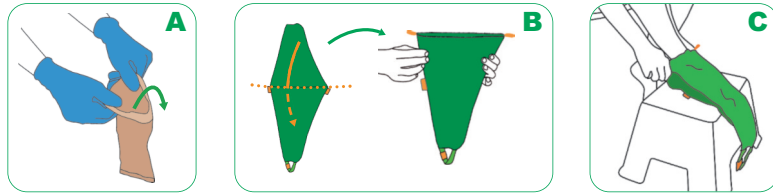
GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

RECOMMENDATION FOR USE

The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

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Arion Slide Solutions B.V.